
INSTRUCTIONS FOR QUARTERLY MAINTENANCE and a POWER OUTAGE

Special Note: Complete this process only if your unit has reached room temperature. Failure to complete this maintenance will result in your unit not operating properly and could cause the handheld modules to become lodged in place.

1. Turn unit off (Model Shown: CT-CT1000)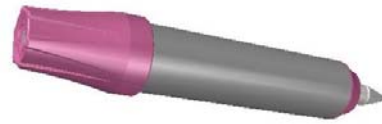

2. After unit has reached room temperature, approximately 8 hours, remove handheld modules from the chilling wells

3. Using the double sided swab

4. Remove all reservoir solution and moisture from chilling wells. Wells should be completely dry.


5. Remove lid and reservoir cup from unit and discard old solution and refill.

6. Dry all handheld modules with cotton towel. Place one full Pipette of reservoir solution in each chilling well. Insert one pen of choice into each chilling well. Place remaining 3 pens in the holding wells.

7. Turn unit on. System will be ready for use in approximately 90 minutes. Test handheld modules (pens) for readiness using the temperature indicator.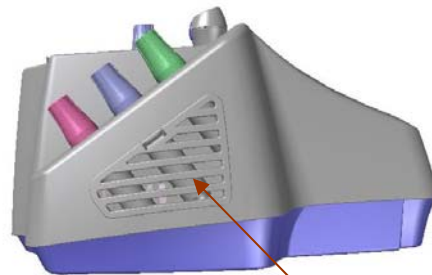

Periodically remove side grill and wipe out inside of unit. Condensation might be visible.

For a Power Surge: Refer to page (4) four of the Operator's Manual. We recommend using a line surge protector to protect your investment.

REFER TO PAGE (8) EIGHT OF THE OPERATOR'S MANUAL FOR ADDITIONAL INFORMATION ON MAINTENANCE.

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